


**EXPORT EXAM (Images & Image Clips):**


Exported exam (images & images clips) are copied not deleted from the exam. Images & image clips can be exported in a variety of image and clip file format or in DICOM format.

- ❖ Only users with appropriate access rights can export exams.
- ❖ Exams exported are not deleted from Q-path.
- ❖ Images and image clips are saved in a ZIP file. Unzip to view the images and image clips.



1. Open the **Exams - List** screen.
2. Select the exam of interest.
3. Select the **Export** button at the top of the screen.

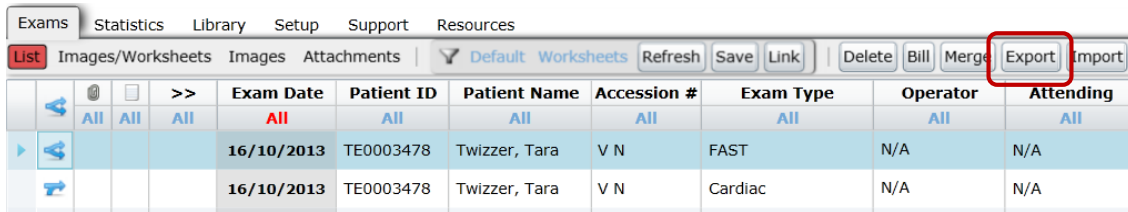

4. A message box pops up prompting you to select file formats.

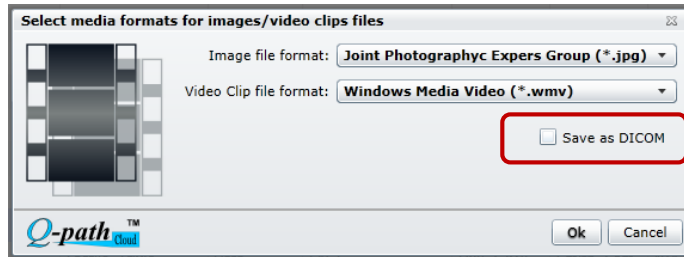

5. Select the **Image file format** and **video clip file format** from the drop down lists.

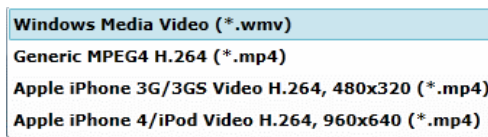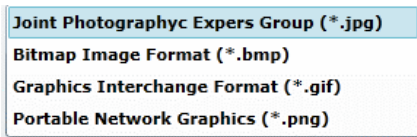

**Or**

Select the **“Save as DICOM”** checkbox (indicated above) to save the exam in DICOM format.


6. Select **OK**. A message from your workstation operating system prompts you to complete the export.
7. Images and image clips are saved in a ZIP file. Unzip to view the images and image clips.

## EXPORT IMAGES/CLIPS:


Images and image clips can be exported for use in teaching or presentations. An individual image or image clip can be exported from either the **Images/Worksheets** or **Images** screen.

- ❖ Images exported are copied not deleted from the exam.
- ❖ Images exported are de-identified if previously configured by your site administrator.


1. Open the exam in either the **Images/Worksheets** or **Images** screen.
2. Select the image or image clip to be exported by clicking the thumbnail image - **do not** select the checkbox beneath the thumbnail image.
  - ❖ The selected image or image clip is highlighted in red (image is displayed in the full image panel to the left).
  - ❖ An image clip will be exported as a video clip file if the clip is active (not paused). If the image clip is paused the current frame will be exported as an image file.
3. Select the **Export** button at the top of the screen.

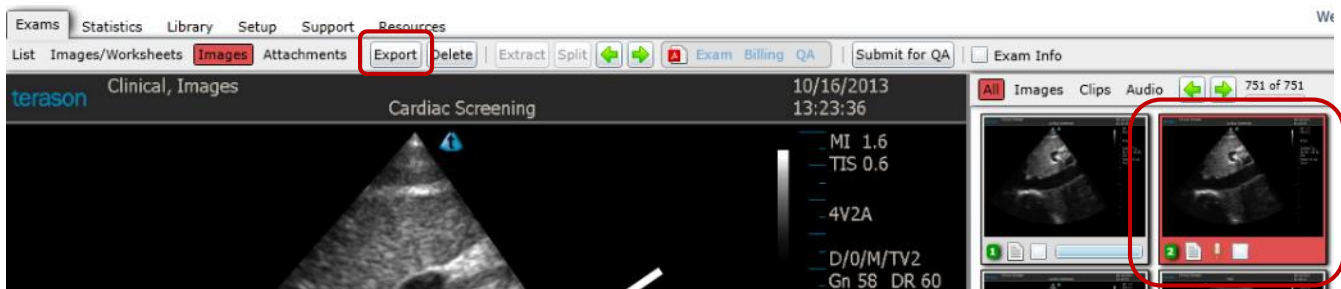

4. A message box pops up as shown below.
  - ❖ The **Image file format** dropdown selection is available when an **Image** has been selected for export or **if a video clip is not active (paused at a select frame)**.

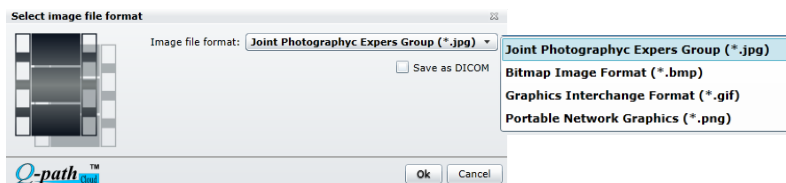

- ❖ The **Video clip file format** dropdown selection is available when an active **Image Clip** has been selected for export.

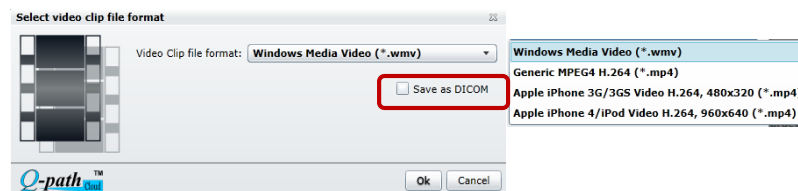

5. Select the file format to export the image, image frame, or image clip from the dropdown menu.  
**Or alternately...**  
Export the image, image frame, or image clip in **DICOM** format by selecting the check box next to **Save as DICOM** (as shown above).
6. Select **Ok**. A message from your workstation operating system prompts you to complete the export.

- ❖ A complete Q-path User Manual is located under the Support tab within your Q-path application